Silagum Comfort

Soft Relining, Primer, Varnish

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Soft Relining: Vinylsilikon/-siloxan, SiO₂, Methylhydrogenpolysiloxan, Additive.

Varnish: Vinylsilikon/-siloxan, Methylsilikon/-siloxan, Additive.

Primer: Ethylacetat, Poly(dimethylsilandiol), Additive.

Instructions for use

English

Product description

Silagum Comfort Soft Relining is a permanently soft relining material for removable dentures, based on A-silicones. Silagum Comfort Primer is a bonding agent for Silagum Comfort Soft Relining. Silagum Comfort Varnish is used to finish dentures relined with Silagum Comfort Soft Relining, creating a smooth surface.

Intended purpose

- Silagum Comfort Soft Relining: Relining material for total, partial and implant anchored dentures
- Silagum Comfort Primer: Bonding agent between denture materials and Silagum Comfort Soft Relining
- Silagum Comfort Varnish: Finishing work on a denture relined with Silagum Comfort Soft Relining

Limitations of use

- Do not use polyether-based impression materials in the case of an indirect relining.
- The impression material should have a long setting time and suitable viscosity.
- Apply Silagum Comfort Primer only to PMMA-based denture bases.

Indication

- Alveolar ridge deterioration
- Irregular alveolar ridge and mucosal contours

Contraindication

- If pre-prosthetic surgery is needed
- Do not use the material if there are known allergies to any of the components or contact allergies.

Patient target group

People who are receiving dental treatment.

Intended user

Dentist, dental technician

Notes for use

- Silagum Comfort Soft Relining: If the material is applied directly in the patient's mouth using a single-use applicator, the single-use applicator must be used only on this one patient for hygienic reasons. For the individual times refer to the "Timing" section.
- Wait at least 24 hours before subjecting the relining to any mechanical stress (e.g. by brushing during cleaning).
- Silagum Comfort Primer and Silagum Comfort Vanish: Always close the bottle tightly to prevent the solvent from vaporizing.
- To achieve an optimum mixing result, use the mixing tips available from DMG. All mixing tips are available as refill packs.

Handling the Automix Cartridge (Fig., see "Handling")

Inserting the Automix Cartridge

1. Push up the lever [A] at the back of the Automix Dispenser and pull back the slide [B] all the way.
2. Lift up the plastic latch [C], insert the Automix Cartridge and lock it using the plastic latch [C].

Note: Make sure that the notches on the Automix Cartridge line up with those on the Automix Dispenser.

Attaching a mixing tip

1. Remove the cartridge cap [D] or used mixing tip after turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

Note: Make sure that the notches on the mixing tip line up with those on the Automix Cartridge.

3. Turn the mixing tip 90° clockwise to lock it.

Applying the material

- When using an Automix Cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- Mix the material in the mixing tip using the lever [F] on the Automix Dispenser and apply directly.

Note: After use, clean the used mixing tip with a standard disinfectant solution and leave it on the Automix Cartridge to act as a cap!

Removing the Automix Cartridge

1. Push up the lever [A] at the back of the dispensing unit and pull back the slide [B] completely.
2. Lift up the plastic latch [C] and remove the Automix Cartridge.

Disinfecting the Automix Dispenser

- The Automix Dispenser can be reused after disinfection, although it should be replaced if there are visible signs of wear.
- Disinfection can be carried out using a commercially available immersion disinfectant. DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute).

Recommended use

Silagum Comfort Soft Relining

Relining performed directly on the patient

1. If required, remove any old soft relining.
2. Grind down the areas to be relined by at least 1 mm. DMG recommends a rounded preparation for the margins of the denture.

Note: Do not contaminate the prosthesis surface with moisture, saliva or monomer-containing plastic resins after these preparations.

3. Apply the primer (Silagum Comfort Primer) evenly over the entire surface in accordance with the manufacturer's instructions and allow the solvent to evaporate completely (approx. 60 s). Ensure that the primer is applied thoroughly to the areas at the edges. Apply at least two layers.
4. Apply Silagum Comfort Soft Relining in a layer at least 2 mm thick, first to the margins of the denture and then to the denture base.
5. Suction out any saliva.

Caution! Triggering of the gag reflex due to uncured silicone.

Note: Ensure that the patient performs the functional movements while sitting upright and leaning slightly forward so that no silicone can make its way into the patient's throat or esophagus.

6. Place the prosthesis into the patient's mouth within the working time and instruct the patient to adopt an habitual occlusion and, with the mouth closed, perform gentle swallowing or chewing movements (without major pressure) for 5 min.
7. Remove the prosthesis from the mouth after 360 s and allow it to cure.
8. Clean and disinfect the prosthesis.

Note: If necessary, make immediate adjustments to the dry, fat-free relining by reapplying and shaping Silagum Comfort Soft Relining. Allow to cure for 420 s in the mouth or 5 min in a pressure pot (40 – 50 °C/104 – 122 °F).

9. Remove excess material with sharp scissors or a scalpel. Smooth the cut edges with a suitable silicone burr.
10. Using a brush, apply Silagum Comfort Varnish to the prepared areas as per the manufacturer's instructions, avoiding air bubbles as much as possible, and allow to dry for 15 min.

Indirect relining at the laboratory

1. Cast the prosthesis with plaster on the jaw side after the functional impression and mount it.

Note: Do not use polyether as an impression material. The impression material should have a long setting time and suitable viscosity. DMG recommends applying an insulating material (plaster against resin).

2. Create a counter by casting the side of the oral prosthesis or the dental arches; the counter must be firmly fixed in its vertical position relative to the cast model. DMG recommends using a special device for relinings with exact repositioning of the cast model and counter (cuvette).
3. Remove the impression material.
4. Grind down the prosthesis surface on the jaw side by at least 1 mm and clean.

Note: Do not contaminate the prosthesis surface with moisture or monomer-containing plastic resins after these preparations.

5. Apply the primer (Silagum Comfort Primer) evenly over the entire surface in accordance with the manufacturer's instructions and allow the solvent to evaporate completely (approx. 60 s). Ensure that the primer is applied thoroughly to the areas at the edges. Apply at least two layers.
6. Apply Silagum Comfort Soft Relining in a layer at least 2 mm thick, first to the margins of the denture and then to the denture base.
7. Set the prosthesis back on the cast model and completely seal the counter using the cuvette.
8. Allow the material to cure for 10 min in the pressure pot at 40 – 50 °C (104 – 122 °F).

Note: If necessary, make immediate adjustments to the dry, fat-free relining by reapplying and shaping Silagum Comfort Soft Relining. Allow to cure for 300 s in the pressure pot (40 – 50 °C/104 – 122 °F).

9. Remove excess material with sharp scissors or a scalpel. Smooth the cut edges with a suitable silicone burr.
10. Using a brush, apply Silagum Comfort Varnish to the prepared areas as per the manufacturer's instructions, avoiding air bubbles as much as possible, and allow to dry.

Silagum Comfort Primer

1. Apply Silagum Comfort Primer evenly to the prepared denture and allow the solvent to evaporate completely (approx. 60 s). Ensure that the primer is applied thoroughly to the areas at the edges. Apply at least two layers.
2. Close the bottle tightly again to prevent the solvent from evaporating.

Silagum Comfort Varnish

1. Mix Silagum Comfort Varnish in a 1:1 ratio until homogenous for 30 s.

- Using a brush, apply Silagum Comfort Varnish to the prepared areas of the prosthesis within the working time, avoiding air bubbles as much as possible.
- Leave Silagum Comfort Varnish to dry for 15 min.
- Close the bottle tightly again to prevent the solvent from evaporating.

Timing

Silagum Comfort Soft Relining	
Working time including mixing time* [s]	≤ 105
Setting time in the mouth [s]	≥ 360
Curing time* [min]	≥ 15

Silagum Comfort Primer	
Air-drying time [s]	≥ 60

Silagum Comfort Varnish	
Mixing time [s]	30
Working time including mixing time* [s]	≤ 90
Curing time* [min]	≥ 15

* The specified times are valid at a room temperature of 23 °C (73 °F) and a normal relative humidity of 50 %. Higher temperatures shorten, lower temperatures lengthen these times.

Interactions

Avoid contact between Silagum Comfort Primer, Silagum Comfort Varnish or the relining material and latex gloves, as these prevent the setting process. In the event of prior contact with latex gloves, wash hands carefully. The use of PVC or PE-based gloves is recommended.

Residual risks/side effects

- Please check the product for damage and visible contamination before use.
- Hypersensitivity and/or allergic reactions cannot be ruled out in individuals who are predisposed.
- A low residual risk of unknown side effects cannot be ruled out.

Warnings/precautions

- For dental use only!
- Avoid contact with eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products used with the material/materials.
- Observe the safety data sheet.
- The manufacturer (productsafety@mtcompanies.com) and the responsible registration authorities must be notified of any serious incidents involving this product.

Product performance characteristics

Complies with ISO 10139-2:2016 Type A soft in combination with Silagum Comfort Soft Relining

Storage/disposal

- Store at 15 – 25 °C (59 – 77 °F) in a dry place.
- Do not store in the refrigerator or freezer.
- Keep away from direct sunlight.
- After use, clean the used mixing tip with a standard disinfectant solution and leave it on the Automix Cartridge to act as a cap!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

Composition

Soft Relining: Vinyl silicone/siloxane, SiO₂, methyl hydrogen polysiloxane, additives.

Varnish: Vinyl silicone/siloxane, methyl silicone/siloxane, additives.

Primer: Ethyl acetate, poly(dimethylsilandiol), additives.

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