# Flairesse Paste

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English

### Product description

Tooth cleaning and polishing paste for professional tooth cleaning and polishing of tooth surfaces as part of a prophylaxis treatment. Flairesse Paste is packaged as a Single-use unit or a tube and is available in various flavors. The three different grit sizes of the pastes enable a graded response to each patient's situation, depending on the level of cleaning required.

### Intended purpose

- Removal of superficial soft and hard plaque.
- Removal of extrinsic stains (e.g. stains from coffee, tea or tobacco).

### Limitations of use

Do not use the material if the patient cannot control the swallowing reflex.

## Indications

- Extrinsic stains
- Tooth plaque

## Contraindications

Do not use the material where there are known allergies to any of the components or contact allergies.

## Patient target group

Persons receiving treatment as part of a dental procedure.

## Intended users

Dentist, dental personnel

## Notes for use

- If the material is applied to the patient's mouth with a single use applicator, the single use applicator must be used for this one patient only for hygienic reasons.
- All disposable prophylaxis contra-angles or polishing cups available on the market may be used.
- DMG recommends using the prophylaxis pastes one after the other (decreasing the grit). Use brushes for occlusal surfaces.
- The Single-use unit and the tube are fitted with a tamper-proof seal.
- Make sure that the tamper-proof seal is intact before the first use.

## Recommended use

**Caution!** Incorrect use may result in overheating.  
► Ensure that there is sufficient material in the polishing cup used.

**Caution!** Risk of swallowing if saliva suction is insufficient.  
► Saliva must be suctioned away thoroughly during treatment.

## Using the paste from the Single-use unit

**Note:** Each Single-use unit holds enough material for one prophylaxis treatment. Each Single-use unit may only be used for one individual patient.

1. Peel off the tamper-proof seal from the Single-use unit using the tab and discard.
2. Insert the Single-use unit into the appropriate prophylaxis holder.
3. To fill, set the polishing cup to low speed and push it into the Prophylaxis Paste to take up the material.
4. Use the polishing cup to apply the Prophylaxis Paste for 20 seconds per tooth at 3000 to 5000 rpm.
5. Suction away excess Prophylaxis Paste and have the patient to spit it out.

## Applying the paste from the tube

**Note:** To prevent the material from drying out, close the tube as soon as you have expressed the material.

1. When using a tube for the first time, unscrew the cap, peel off the tamper-proof seal using the tab and discard.
2. Express a sufficient quantity of the material onto a mixing pad.
3. To fill, set the polishing cup to low speed and push it into the Prophylaxis Paste to take up the material.
4. Use the polishing cup to apply in the Prophylaxis Paste for 20 seconds per tooth at 3000 to 5000 rpm.
5. Suction away excess Prophylaxis Paste and have the patient to spit it out.

## Interactions

No interactions are currently known.

## Residual risks / side effects

Swallowing excessive amounts of the material may cause nausea. The residual risk of hypersensitivity to components of the material cannot be ruled out.

## Warnings / precautions

- For dental use only.
- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.
- Do not swallow.

## Performance characteristics of the device

Fluoride content: approx. 12,300 ppm

## Storage / disposal

- Store in a dry place at 15 to 25°C (59 to 77°F)!
- Do not store in the refrigerator.
- Keep away from direct sunlight.
- Do not use after the expiration date.
- Disposal must comply with national regulations.

## Composition

Pumice (approx. 47 %), glycerol (approx. 35 %), water (approx. 9 %), sodium fluoride (< 3 %), phosphoric acid (< 2 %), additives (approx. 5 %)  
Free from aspartame, gluten and saccharin.  
Contains xylitol.

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