# Flairesse Varnish

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English

### Product description

Moisture and saliva tolerant varnish for treating hypersensitive teeth and to be applied to the enamel and dentin as part of a prophylactic treatment. Flairesse Varnish is packaged as a Single-use unit and is available in two different flavors.

### Intended purpose

- Treatment of hypersensitive teeth
- Sealing of hypersensitive cervices and exposed root dentin
- Protective varnish for the prevention of caries
- Remineralization of incipient caries

### Limitations of use

Do not use the material if the patient cannot control the swallowing reflex.

### Indications

- Hypersensitivity of the teeth
- Postoperative sensitivities
- Incipient caries
- Teeth with a high caries risk

## Contraindications

- Do not use the material if the patient's gums are ulcerated.
- Do not use the material if the patient is suffering from stomatitis.
- Do not use the material where there are known allergies to any of the components or contact allergies.

## Patient target group

Persons receiving treatment as part of a dental procedure.

## Intended users

Dentist, dental personnel

## Notes for use

- If the material is applied to the patient's mouth with a single use applicator, the single use applicator must be used for this one patient only for hygienic reasons.
- The Single-use unit is fitted with a tamper-proof seal. Make sure that the tamper-proof seal is intact before the first use.

## Recommended use

**Note:** Each Single-use unit holds enough material for one prophylaxis treatment. Each Single-use unit may only be used for one individual patient.

1. Remove calculus and plaque.
2. Remove saliva and fluid from the areas to be treated.
3. Peel off the tamper-proof seal from the Single-use unit using the tab and discard.
4. Take the brush and use it to carefully mix the varnish.
5. Bend the brush as required for treatment.
6. Apply a thin, even coating of the varnish to the areas to be treated.
7. Allow to act for 4 hours.

**Note:** In order to allow the varnish to act over 4 hours, inform the patient that the following should be avoided for this period:

- solid food
- alcoholic drinks
- alcohol-based oral rinses
- tooth brushing
- dental floss

## Interactions

- Other prescription products containing fluoride should not be used within 24 hours of using the varnish.
- Patients using regular, systemic fluoridation should stop using these products for 2 or 3 days following use of this varnish.

## Residual risks / side effects

- In rare cases, use of the product may lead to edematous difficulties in patients suffering from asthma.
- In rare cases, extensive application may lead to oedematous swelling of the gums.
- Swallowing excessive amounts of the material may cause nausea.
- The residual risk of hypersensitivity to components of the material cannot be ruled out.

## Warnings / precautions

- For dental use only.
- Keep out of the reach of children!
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.
- Do not swallow.

## Performance characteristics of the device

Fluoride content: approx. 22600 ppm

## Storage / disposal

- Store in a dry place at 20 to 25 °C (68 to 77 °F)!
- Do not store in the refrigerator.
- Keep away from direct sunlight.
- Do not use after the expiration date.
- Disposal must comply with national regulations.

## Composition

Natural resin (rosin, approx. 59 %), ethanol (approx. 27 %), sodium fluoride (approx. 5 %), additives (approx. 10 %)

Free from aspartame, gluten and saccharin.

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